

Model 13 Vertical Stabiliser Strap

For extra stability - If required



This simple vertical stabiliser strap can be fitted to the seat back to offer extra stability - by restricting excess movement. As the buckles are positioned at the rear, it can be left in position on the seat without having to remove it each journey.

Fitting Instructions

With the fixing strap fitted vertically around the vehicle seat (as in the images above) - fit the Model 13 harness as follows:

1. Place the PVC chest pad against the patient's chest.
2. Pass both horizontal fixing straps around the patient and fasten straps behind the seat back. The lower strap at waist level and the top strap at chest level to hold the patient securely in an upright sitting position.
3. Pass the shoulder straps over the patient's head and shoulders so that the tongue end of the Steel Safety Buckle is hanging down behind the seat back. The shoulder straps can be adjusted in length so that the buckle can be positioned well away from their head and out of sight and out of reach of the passenger.



4. Connect the shoulder strap buckle to the top buckle of the Stabiliser Strap (See image A).



5. Pass the understrap between the patient's knees, down underneath the base of the seat and up behind the seat back. Connect to the lower Stabiliser Strap Buckle (See image B).

6. Adjust all straps to fit securely yet comfortably.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com