

Houdini Stop Chest Strap



The Houdini Stop chest strap ensures the passenger stays safely and correctly positioned within their harness. It simply clips on to each shoulder strap of the harness neatly **positioned underneath the adjusters** to help prevent the passenger from separating the shoulder straps in order to escape.

Once the Houdini Stop chest strap is in place, it should stay there. The passenger can not easily move it up or down. It is universal and can be used on many types of harnesses, from child car seats (if approved by the car seat manufacturer), H Harnesses, highchairs, and even prams or strollers.

The Houdini Stop is a very useful training product for the stage where your child does not understand the importance of staying inside their harnessing. It suits many age ranges and is simple for an adult or older child to remove if necessary.

The Houdini Stop has been officially crash tested to the **AS/NZ 1754 Standard by Autoliv Australia** and independently reviewed by ACRI in Australia. This revolutionary car seat chest clip has been proven not to interfere with the existing car seat harnessing in anyway and does not ride up and cause a choking hazard in a simulated accident situation.

If you are intending to use this strap on a child car seat we would strongly advise that you **check with the car seat manufacturer first to seek their approval** as there can be differences in regulations between EC and other countries.
(NB: Britax have not approved the use of the Houdini Stop for use with their child car seats)

The Houdini Stop can be fitted around the shoulder straps of our harnesses to hold them together and help prevent them from slipping off the shoulders. This can make all the difference to the success of some harnesses. In particular, the designs such as Model 19, 27 and 28, where there can be some tendency for the more active passenger to try and remove the straps from off their shoulders in order to "escape".

WARNING

Chest straps must never be used with chest padded harnesses as the strap would be positioned too close to the neck area.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com