

Model 13 Wheelchair Version

The Model 13 is a fully rear fastening harness designed for those passengers with challenging behaviour and or learning difficulties when seated on a single vehicle seat.

It also provides postural support & assists in seating the occupant in an upright sitting position.

The wheelchair version is fitted with three lightweight Side Squeeze buckles so it is more suitable for wheelchair use.


This harness can also be used on some easy chairs.

IMPORTANT: This harness is not classed as a safety harness.

When travelling in a wheelchair in transit, the vehicle safety belt **MUST** be worn over the top of our harness unless they are exempt for medical reasons. The combination on the existing safety belt & the secondary harness offers added security for the passenger.



50mm Side Squeeze Buckle

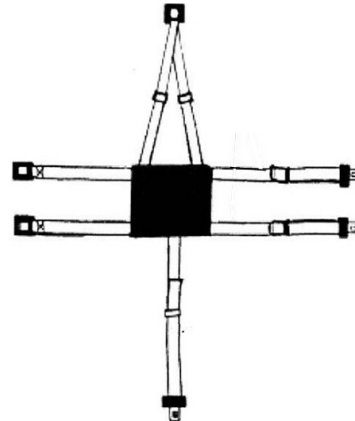

Available in two sizes:

Up to 6 years - Cat. Ref: 13WJ

7 years to Adult - Cat. Ref: 13W

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com

Model 13 Fitting Instructions

1. Place the PVC chest pad against the patient's chest.
2. Pass both horizontal fixing straps around the patient and fasten straps behind the seat back. The lower strap at waist level and the top strap at chest level to hold the patient securely in an upright sitting position.
3. Pass the shoulder straps over the patient's head and shoulders so that the female / cup end of the Side Squeeze Buckle is hanging down behind the seat back. The shoulder straps can be adjusted in length so that the buckle can be positioned well away from their head and out of sight and out of reach of the passenger.
4. Pass the understrap between the patient's knees, down underneath the seat base and up behind the seat back to connect to the shoulder strap buckle.
- 5 Adjust all straps to fit securely yet comfortably.



If you should need further advise about this product please do not hesitate to contact us.

IMPORTANT: BELTS AND HARNESES SHOULD BE REPLACED IF THEY SHOW ANY SIGN OF FRAYING OR CUTS, AND ESPECIALLY IF THEY HAVE. BEEN INVOLVED IN AN ACCIDENT.